



The Intelligent Cardiovascular Ultrasound Scanner

27 September 2017

By Erik N. Steen, Chief Engineer GEHC Cardiovascular Ultrasound

This presentation partly describes ongoing research and development efforts. These efforts are not products and may never become products

Cardiologist

How can I be confident in my ability to manage my patient's heart health when 10-15% of the patients have suboptimal echoes?



Interventional Cardiologist

I need a better understanding of the anatomy and function during structural heart repairs





Vivid™ E95

Cardiovascular Ultrasound
with



cSound™



cSound™


Intelligent processing

- Channel data from many transmits collected into GPU memory in real time
- Image is computed in real time by software algorithms
- High performance
- Great flexibility to change algorithms

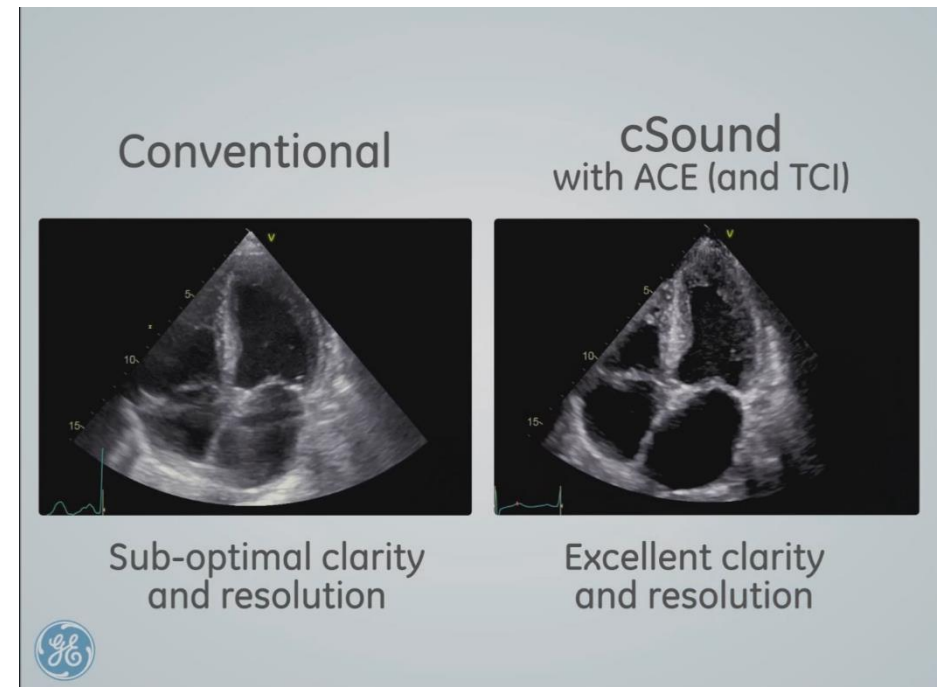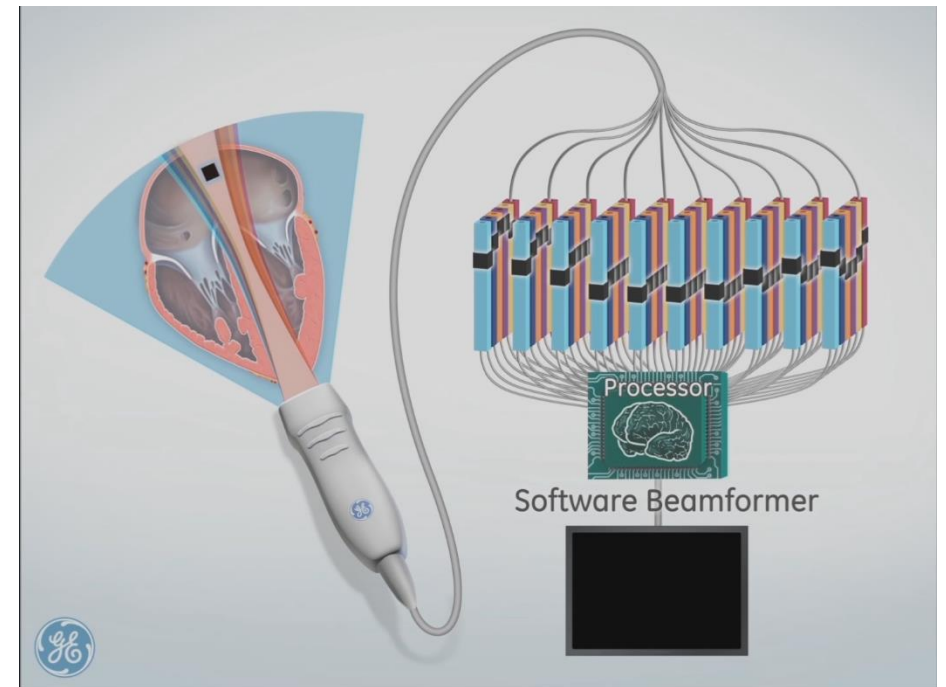


HDlive™

Examples from interventions

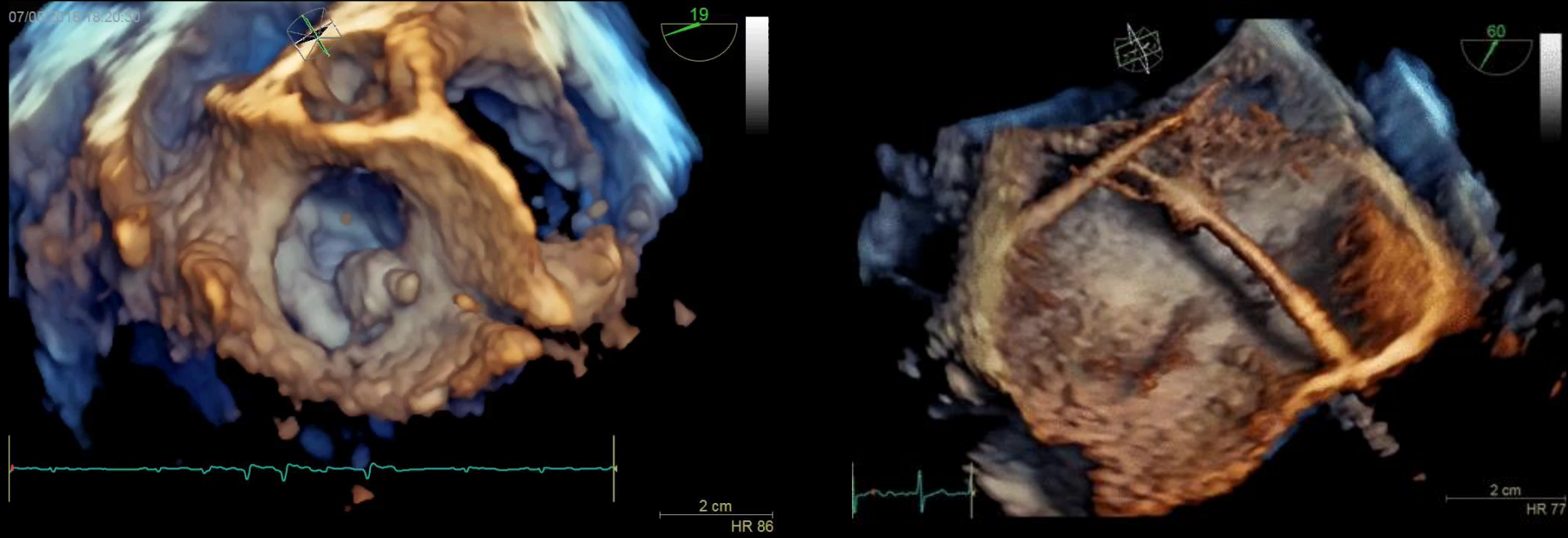


With cSound™, image reconstruction algorithms can be changed according to clinical needs

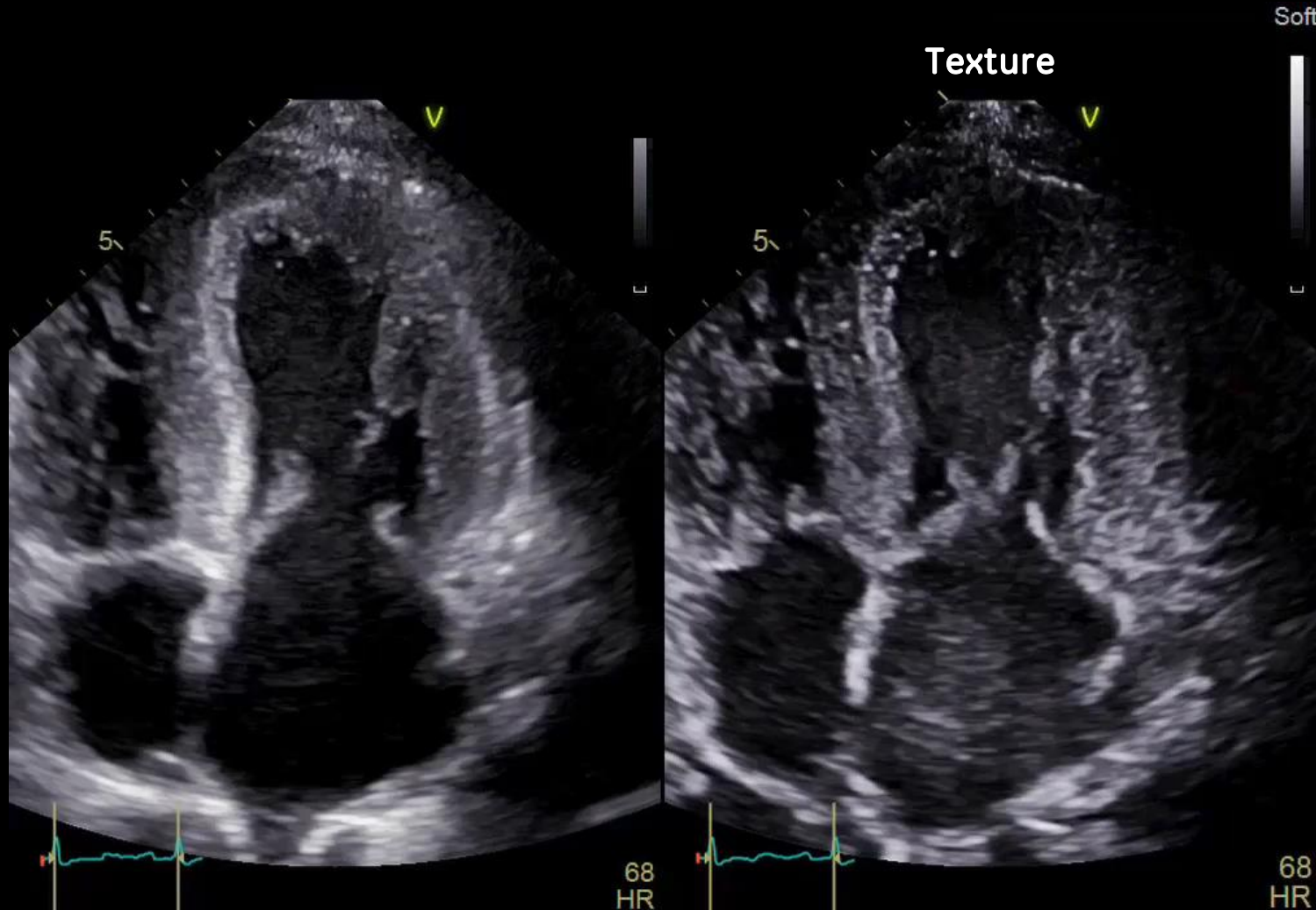

Amyloidosis example (ACE+TCI vs Texture)



Blood flow can be visualized in completely new ways

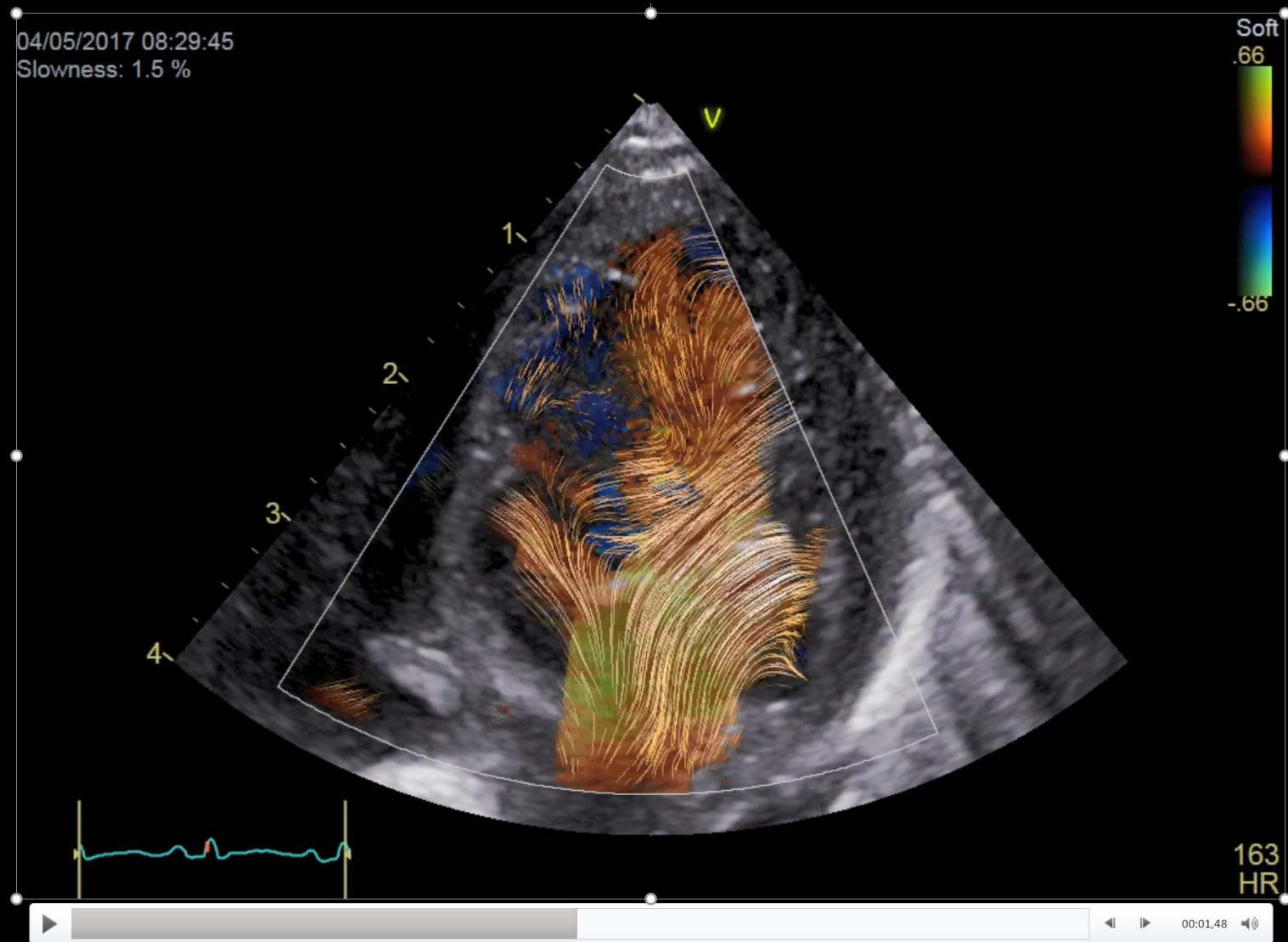

Cardiologist

How can I be confident in my ability to manage my patient's heart health when 10-15% of the patients have suboptimal echoes?


I need a better understanding of the anatomy and function during structural heart repairs

How can I become more efficient with the increased burden of cardiovascular disease and pressure on cost ?



Automatic Doppler Measurements

Performing manual Doppler measurements (tracings) is time consuming

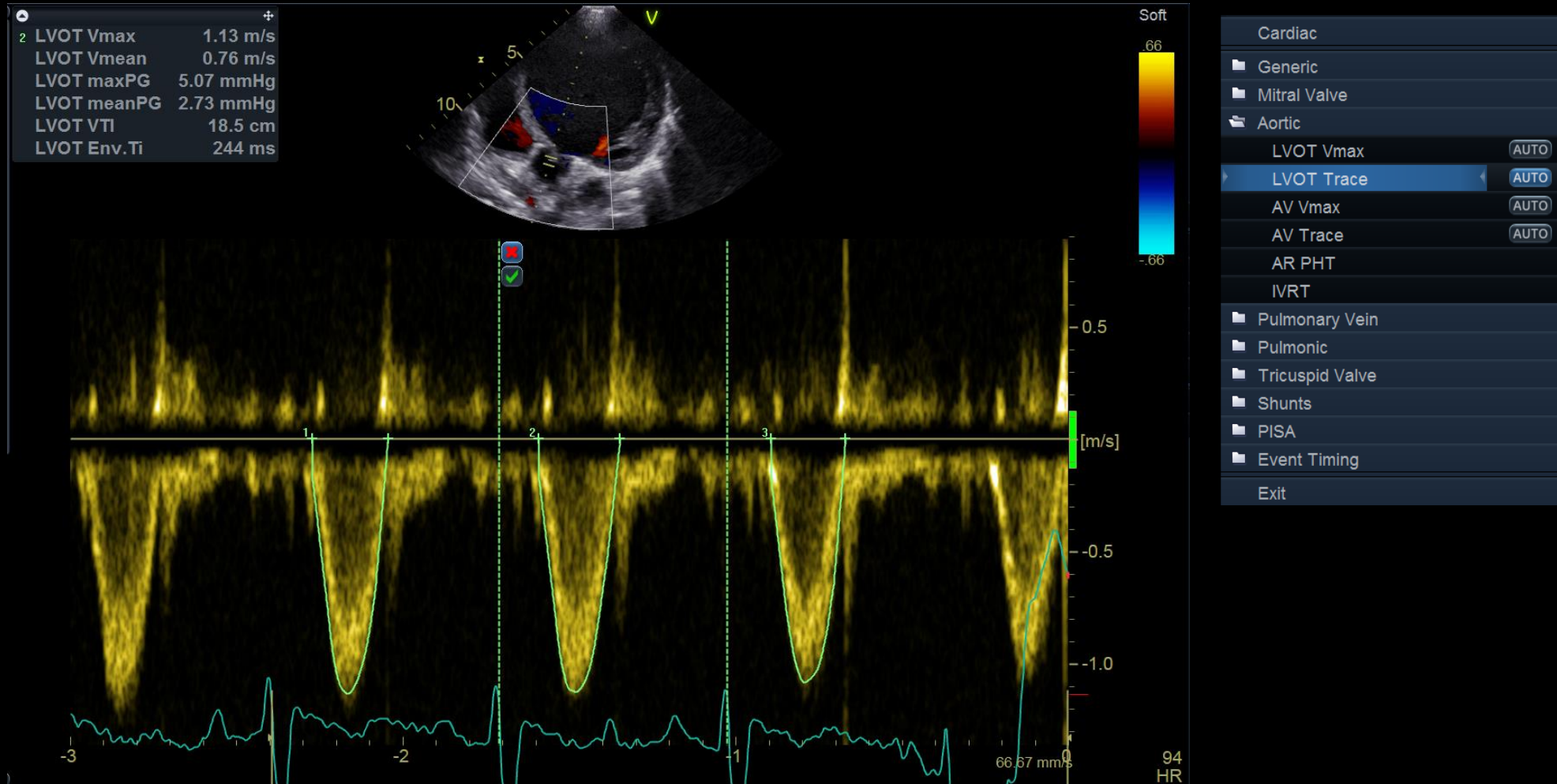

Active for the most common

measurements:

- LVOT Vmax
- LVOT Trace
- AV Vmax
- AV Trace
- TR Vmax
- MV E/A Velocity
- E'

Auto Doppler may **reduce scan time**, improve **consistency** (less user dependent) and eventually make the exam more **efficient**



Cardiologist

How can I be confident in my ability to manage my patient's heart health when 10-15% of the patients have suboptimal echoes?

I need a better understanding of the anatomy and function during structural heart repairs


How can I become more efficient with the increased burden of cardiovascular disease and pressure on cost ?

How can I make sure my cardiovascular ultrasound system is future-proof ?







Future development(*)

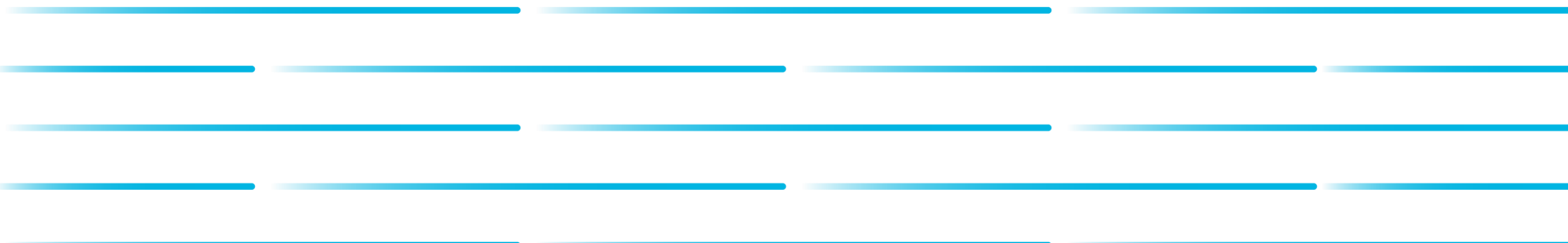

**Note: Technology in development that represents ongoing research and development efforts. These technologies are not products and may never become products. Not for sale. Not cleared or approved by the U.S. FDA or any other global regulator for commercial availability*

Faster GPUs can be combined with new algorithms for greatly improved 3D imaging (*)


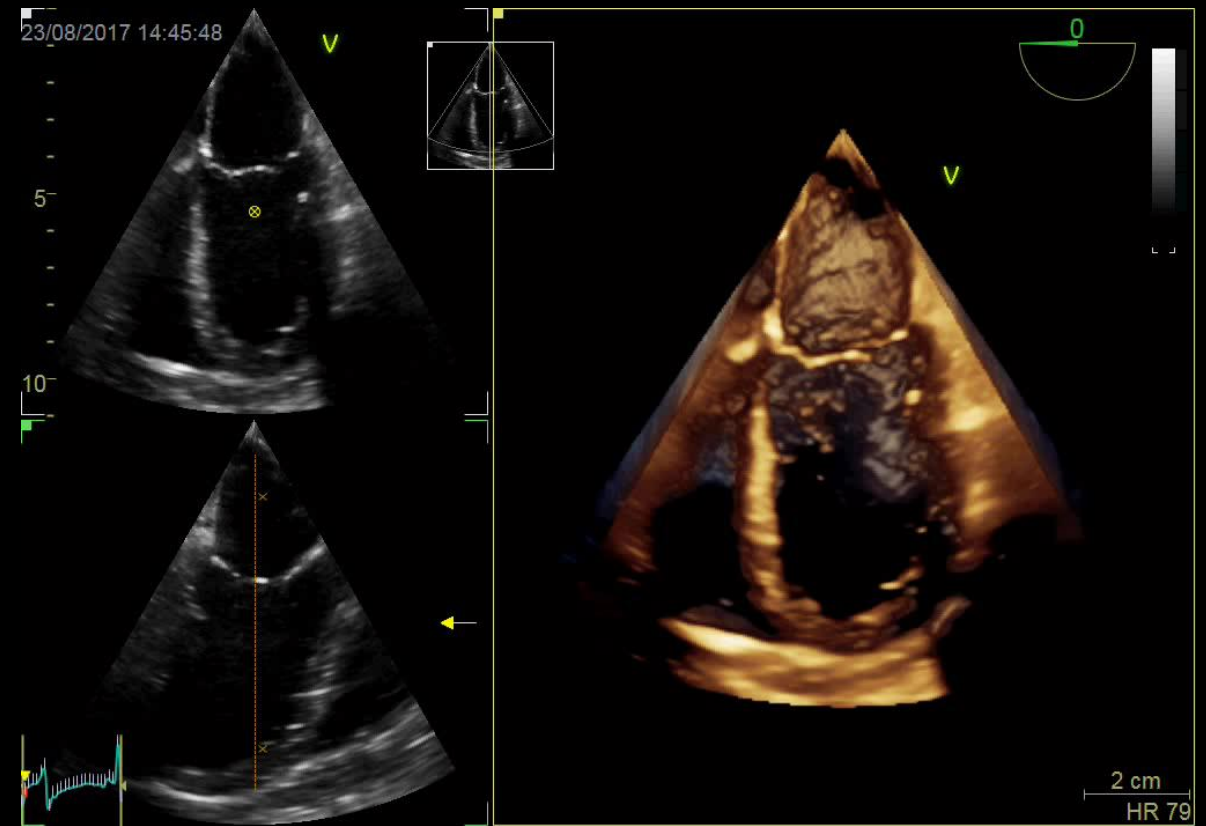
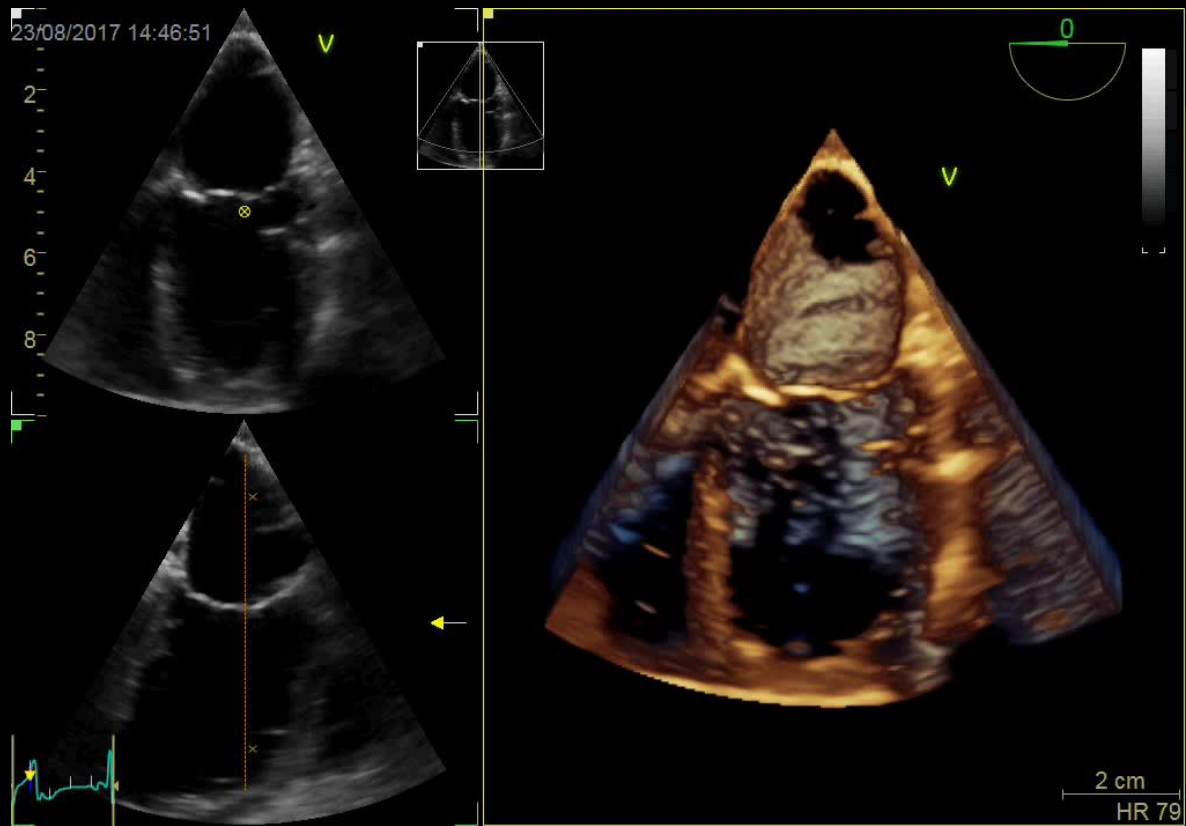


**Note: Technology in development that represents ongoing research and development efforts. These technologies are not products and may never become products. Not for sale. Not cleared or approved by the U.S. FDA or any other global regulator for commercial availability*

cSound™


Intelligent workflow

- Workflow is automatically optimized according to cardiac view.

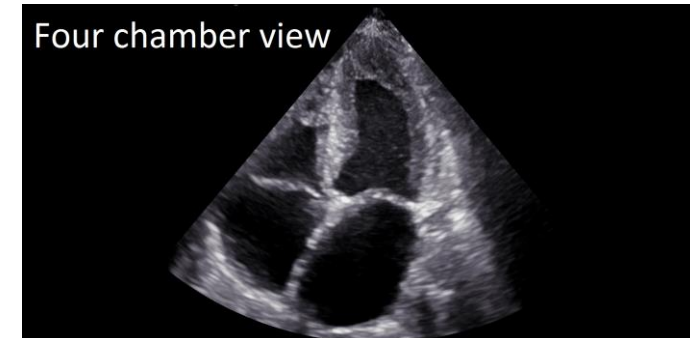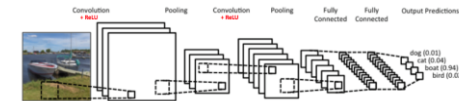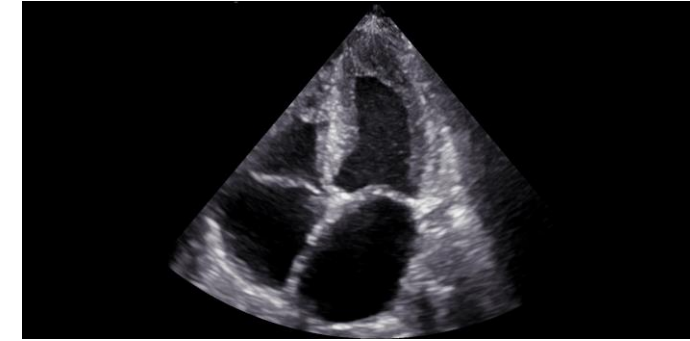

**Note: Technology in development that represents ongoing research and development efforts. These technologies are not products and may never become products. Not for sale. Not cleared or approved by the U.S. FDA or any other global regulator for commercial availability*

Automatic Cardiac View Recognition (*)

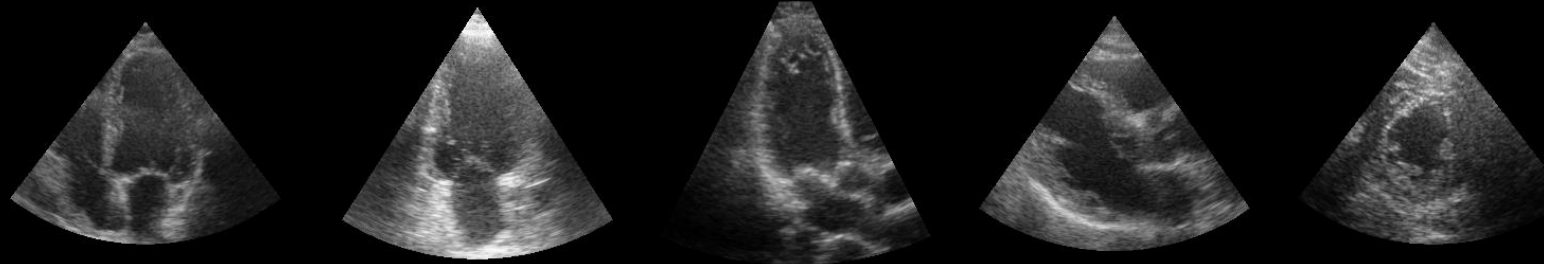

Apical 4 chamber view

Apical 2 chamber view

Apical long axis view

Parasternal long axis view

Parasternal short axis view

Preliminary results

- **Data: >100.000 images from > 5000 loops with variable image quality & patient anatomy**
- **500 loops used for validation**
- **Various network architectures investigated**
- **Accuracy (CaffeNet): 96 % accuracy on frame level, 97 % accuracy on sequence level (using majority vote)**
- **< 2 ms per frame inference time using a Quadro P4000 GPU**



**Note: Technology in development that represents ongoing research and development efforts. These technologies are not products and may never become products. Not for sale. Not cleared or approved by the U.S. FDA or any other global regulator for commercial availability*